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### EN Minimum System Requirement for Computer Imaging Systems

The following information simply covers the minimum computer system requirements for imaging purposes. Smooth operation of Dürr Dental hardware and software can be affected through specific hardware and software incompatibility on site at the client despite meeting system requirements. In this case, Dürr Dental cannot guarantee smooth and trouble-free operation of the Dürr Dental hardware and software. Where further systems are connected to the PC please note that this may change the system requirements. Observe the system requirements for all connected systems.

		VistaCam CLIX	VistaCam Digital	VistaCam IX	VistaScan Mini	VistaScan Mini Plus	VistaScan Mini View	VistaScan Perio	VistaScan Perio Plus	VistaScan Perio Net	VistaScan Combi	VistaScan Combi Plus	VistaScan Combi View	VistaRay 6	VistaRay 7	DBSWIN ab Version 5.4	VistaEasy	Image Bridge
<b>CPU:</b>	Intel Pentium IV compatability, 1.4 GHz or better, neither Celeron nor Duron	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x
<b>RAM:</b>	1 GB (2 GB recommended)	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x
<b>Operating System:</b>	Microsoft Windows XP Professional, 32 bit, Service Pack 3	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x
	Microsoft Windows 7, 32 bit (from Home Premium)	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x
	Microsoft Windows 7, 64 bit (from Home Premium)	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x
	Microsoft Windows 8, 64 bit (not Windows RT)	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x
<b>Hard disk:</b>	Workstation (not incl. data bank) ≥10 GB	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x
	Data bank DBSWIN hard disk > 100 GB																	
	 The exact memory requirements should be calculated on the basis of the number and quality of Images taken in the particular surgery.																x	
<b>Drive:</b>	DVD-ROM	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x
<b>Data back-up:</b>	For daily data back-up recommend: tape drives e.g (DAT, DITTO, TRAVAN, ADR (OnStream))				x	x		x	x	x	x			x	x			x
<b>Mainboard:</b>	Intel-Chipset / Via-Chipset	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x
	Nvidia-Chipset / AMD-Chipset	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x
<b>Interface:</b>	USB 2.0 Interface	x	x	x	x	x		x	x		x	x		x	x			
	USB 3.0 Interface		x <sup>2</sup>	x	x	x		x	x		x	x		x	x			
	Parallel Interface, EPP mode IEEE 1284 or										x	x						
	Ethernet ≥100 Mbit				x	x	x		x	x	x <sup>3</sup>	x <sup>3</sup>	x				x	x
<b>Graphics:</b>	Minimum resolution 1024 X 768	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x
	Minimum colour depth 32 bit, 16.7 million colours	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x
<b>Monitor:</b>	According to radiation regulation, SVGA, min. 17" (43 cm), ≥ 1024 x 768 Pixel, 24 / 32 Bit colour Intensity				x	x	x	x	x	x	x	x	x	x	x			
<b>Cable length not to exceed:</b>	USB: 5 m				x	x		x	x		x	x		x	x			
	additional 3x USB repeater (max 5 m each) possible	x <sup>1</sup>	x <sup>1</sup>	x <sup>1</sup>	x <sup>1</sup>	x <sup>1</sup>		x <sup>1</sup>	x <sup>1</sup>		x <sup>1</sup>	x <sup>1</sup>						

<sup>1</sup> only with active USB 2.0 or USB 3.0 Hub

<sup>2</sup> since year of production 2008

<sup>3</sup> optionally with VistaScan Net (order number 2130-550-00)

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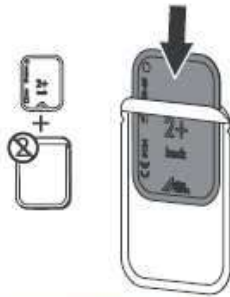
### Short introduction

The following information is supplementary to the appliance Installation and Operating Instructions.


Always refer to the Installation and Operating Instructions of the appropriate appliance in which important information is detailed, e. g. Safety Instructions, Set-up, Electrical Connections, Disinfection, Cleaning, etc.

#### 1 Reading the image data using the computer - intraoral


1.

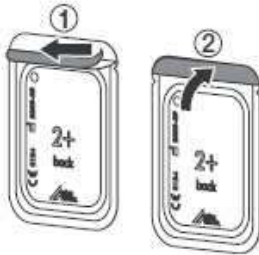
2.

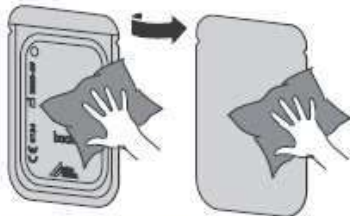
3.

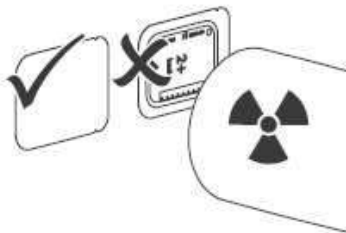
4.

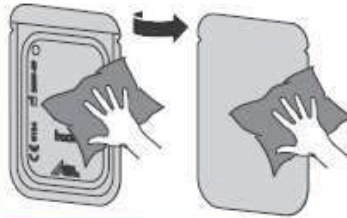5.

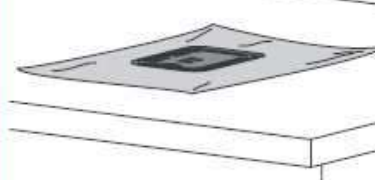6.


7.

8.

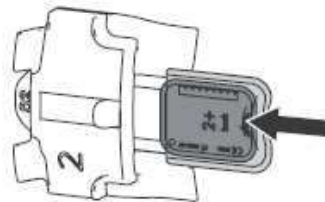
9.


10.

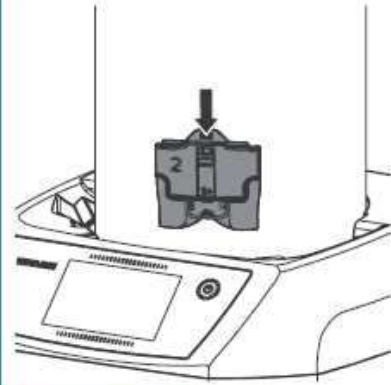
11.

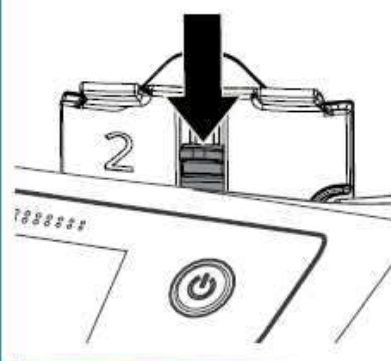12.

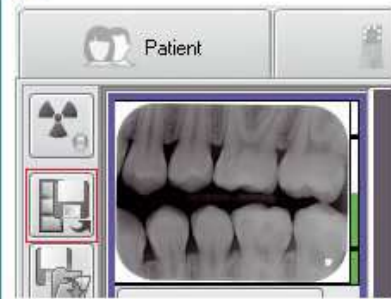13.




Short introduction

The following information is supplementary to the appliance installation and Operating Instructions.

Always refer to the Installation and Operating Instructions of the appropriate appliance in which important information is detailed, e. g. Safety Instructions, Set-up, Electrical Connections, Disinfection, Cleaning, etc.

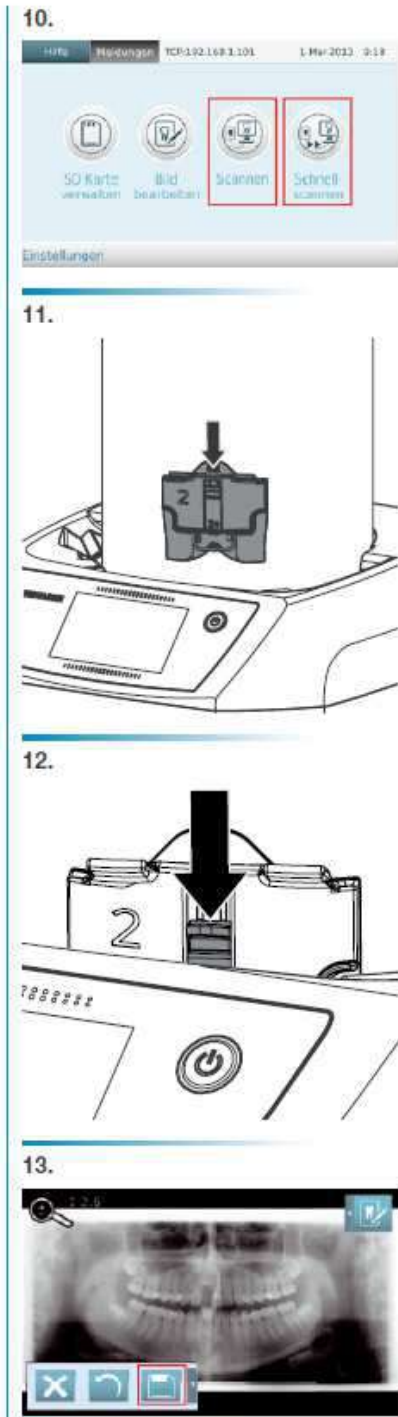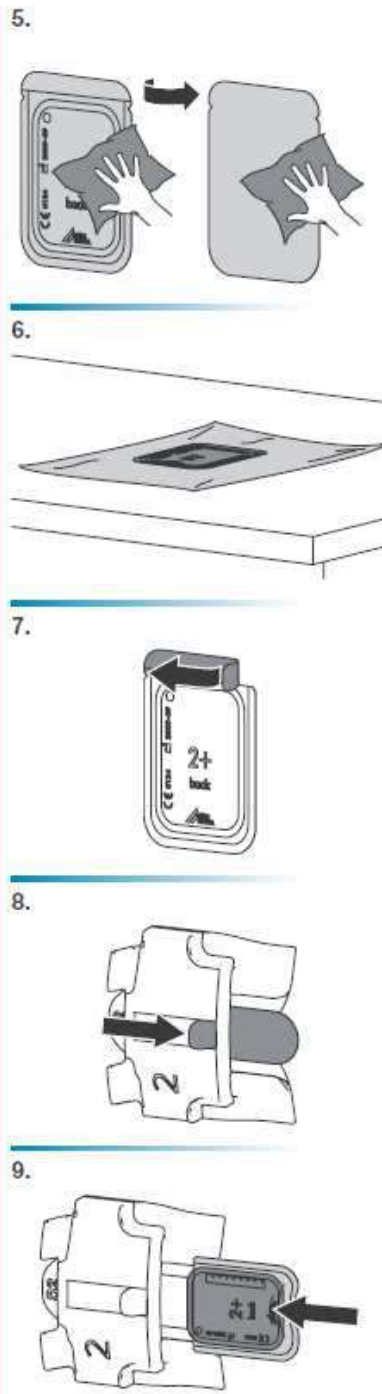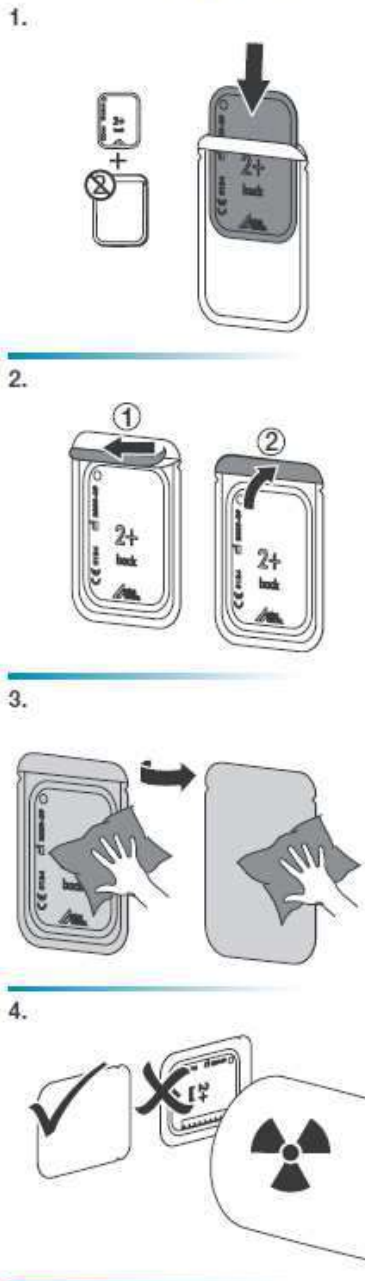2 Scanning image data using the unit touch screen - intraoral

DÜRR DENTAL AG  
Höpfelheimer Strasse 17  
74321 Bietigheim-Lissingen, Germany


1303V001


**3 Reading the image data using the computer - extraoral**

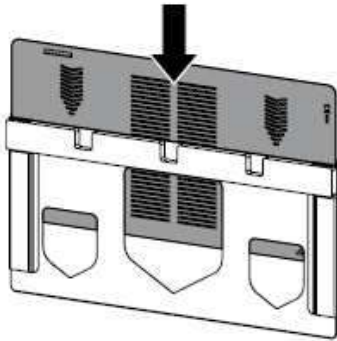
1.

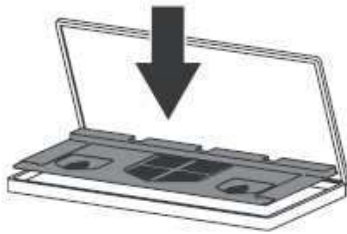2.

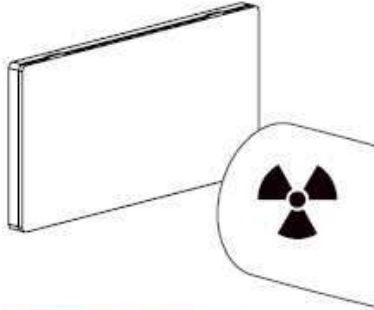
3.


4.

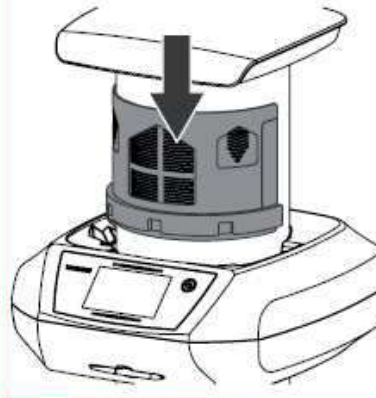5.

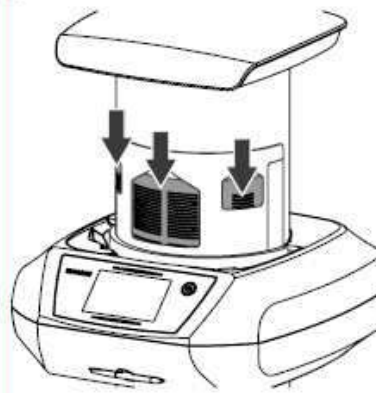6.


7.


**4 Scanning image data using the unit touch screen - extraoral**

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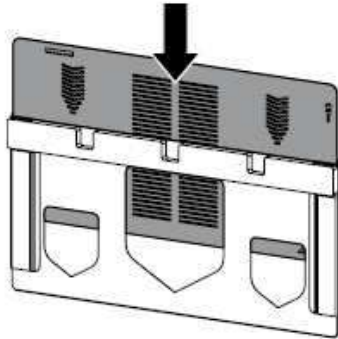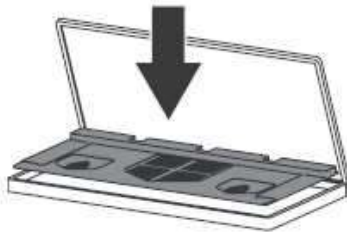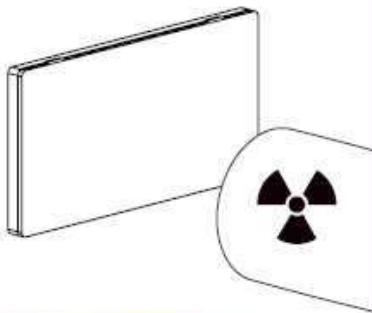
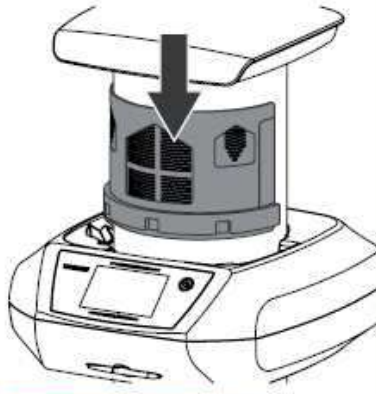
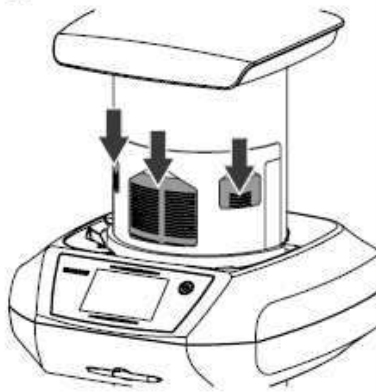
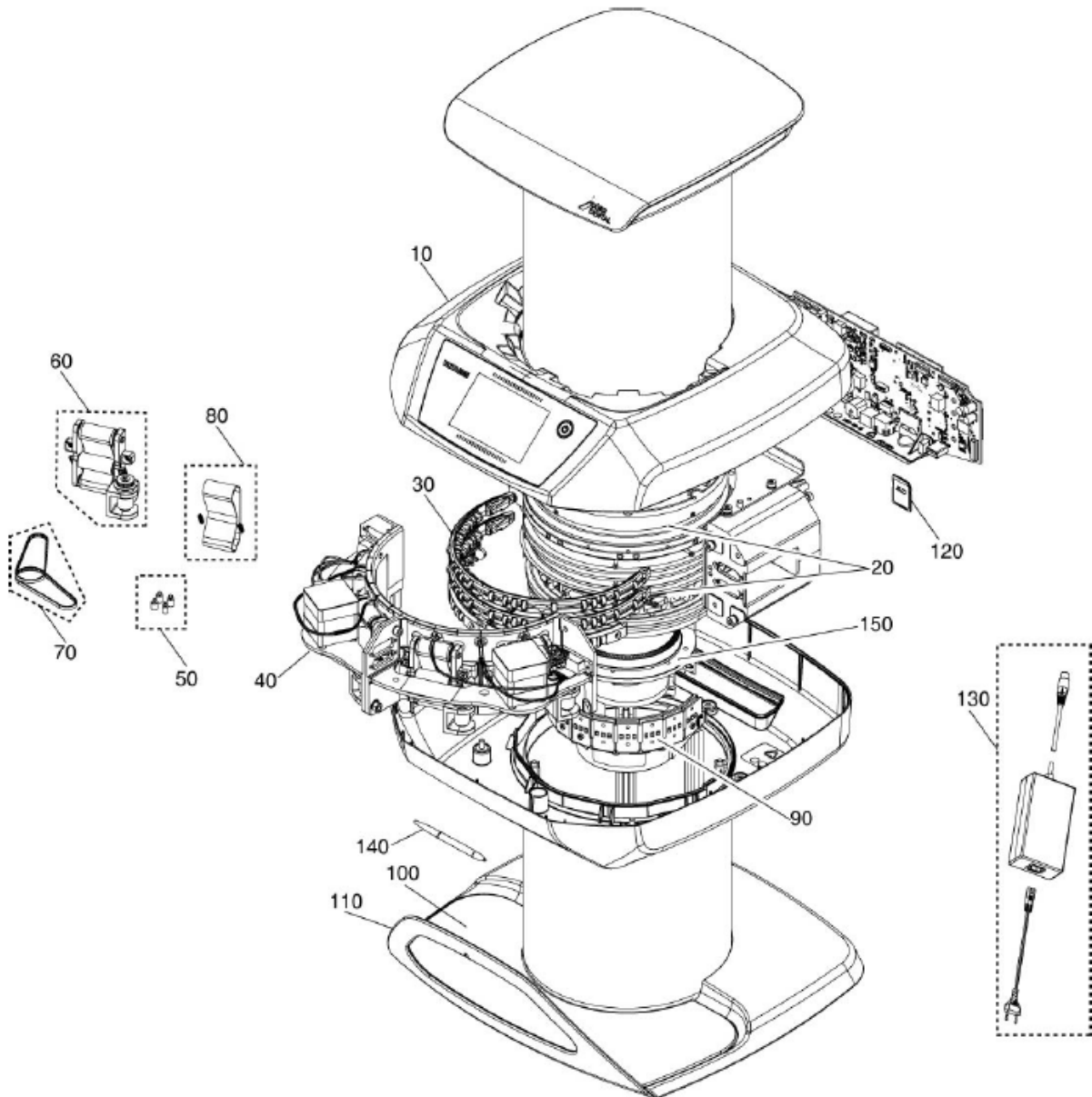
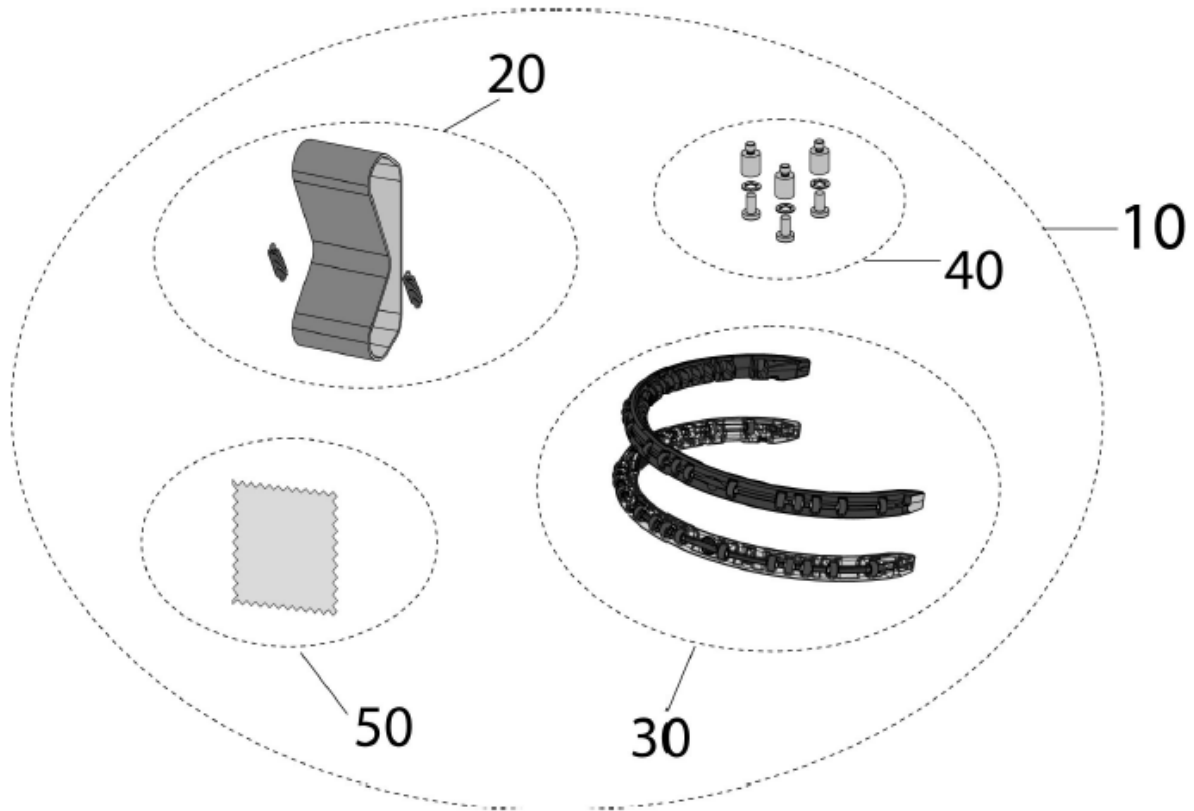
### Kit maintenance VISTASCAN Combi View

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Code	Text	example or solution
-1	ERR_FILE_NOT_FOUND	File not found
-2	ERR_PARAM	Parameter not found or parameter checking failed
-2	Parameter not found or parameter checking failed	
-3	ERR_INIT_FAIL	Error while writing the configuration data
-4	ERR_DEVICE_NOT_OPEN	Device or file was not opened for access
-5	ERR_INVALID_CTRL_CODE	Ctrl code is not valid
-6	ERR_IS_ACTIVE	Activate is called without a deactivate beforehand
-7	ERR_IS_OPEN	Scanner is already open
-8	ERR_MEMORY	No memory could be allocated
-9	ERR_DEV_TYPE	WrongType of Scanner
-10	ERR_DEV_TYPE_CHANGE	Change VistaScan / PerioScan detected
-11	ERR_FAIL	general Error
-12	ERR_DLL	dll or function in dll is missing
-13	ERR_STUB	function / method implementation missing - stub only
-14	ERR_INVALID_CALL	function parameter check failed, e.g. invalid pointer
-15	function parameter checking failed	
-16	ERR_WRITE	file io error, e.g. locked or write failed
-17	ERR_DOESNT_EXIST	scanner or location doesn't exist
-18	ERR_EXISTS	scanner or location already exist
-19	ERR_NOT_EMPTY	dir is not empty: device exists in location to be removed
-20	ERR_VALUE_NOT_IN_RANGE	initialisation parameter not in valid range
-21	ERR_MISSING_CAPABILITIES	Addressed device lacks some features
-22	ERR_BUSY	device or function busy
-23	ERR_DLL_INIT_FAILED	library or dll init failed - component missing / setting or file not found
-24	ERR_VERSION	version of (sub-)component does not match the required version
-50	ERR_INVALID_IMAGE	Image struct contains inconsistent data or doesn't exist in memory
-61	ERR_NO_INI_SECTION_OR_KEY	neither Section nor Key exist
-62	ERR_NO_INI_SECTION	Section does not exist
-63	ERR_NO_INI_KEY	Key does not exist

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-402	VSU_STATUS_REG_OPEN (	
-402		VSCAN USB
-403	VSU_STATUS_READ (	
-403		VSCAN USB
-404	VSU_STATUS_WRITE (	
-404		VSCAN USB
-406	VSU_STATUS_IHEX_ERROR (	
-406		VSCAN USB
-407	VSU_STATUS_IHEX_CRC_ERROR (	
-407		VSCAN USB
-500	electricallay conflict on the UBS-Bus	VSCAN USB
-500	VSU_STATUS_I2C_BUS_ERR (	
-501	slave NAKs after send START and the slave address	VSCAN USB
-501	VSU_STATUS_I2C_START (	
-502	slave NAKs after write a second parameter	VSCAN USB
-502	VSU_STATUS_I2C_WRITE (	
-503	VSU_STATUS_I2C_READ (	
-503		VSCAN USB
-504	slave NAKs after write EEPROM address	VSCAN USB
-504	VSU_STATUS_I2C_PROM_ADR (	
-506	timeout wait for EEPROM write (program EEPROM)	VSCAN USB
-506	VSU_STATUS_I2C_PROM_PROG (	
-507	error during EEPROM read	VSCAN USB
-507	VSU_STATUS_I2C_PROM_READ (	
-508	false parameter in the request , request is second param.	VSCAN USB
-508	VSU_STATUS_USB_REQ_PARAM (	
-509	false length parameter in setup request	VSCAN USB
-509	VSU_STATUS_USB_REQ_LEN (	
-510	error during execute request	VSCAN USB
-510	VSU_STATUS_USB_REQ_EXECUTE (	
-511	error during reset wait for donw bit	VSCAN USB
-511	VSU_STATUS_RESET_FPGA (	
-512	error during write to FPGA	VSCAN USB
-512	VSU_STATUS_WRITE_FPGA (	
-600	ERR_WRONG_IMAGETYPE	this image operation is not available for the given image type (most ops are only available for 16Bit).

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-601	<b>ERR_UNKNOWN_IMAGETYPE</b>	Imagetype for the imagefile could not be retrieved.
-601	Imagetype for the imagefile could not be retrieved.	DD Bibliotheksfehler
-602	<b>ERR_WRONG_IMAGE_FILE</b>	this operation is not available for this image file.
-602	this operation is not available for this image file.	DD Bibliotheksfehler
-987	<b>VSU_STATUS_UNEXPECTED (</b>	
-987		VSCAN USB
-998	<b>VSU_STATUS_FUNCTION_NOT_COMPLETED (</b>	
-998		VSCAN USB
-999	<b>VSU_STATUS_FUNCTION_NOT_IMPLEMENTED (</b>	
-999		VSCAN USB
-1000	<b>ERR_INTERFACE_BUSY</b>	LPT interface already in use \todo germish
-1000	LPT interface already in use	an other software uses the LPT port
-1000	LPT interface already in use \todo germish	
-1001	<b>ERR_INTERFACE_CONNECTION</b>	Scanner not found
-1001	Scanner not found	driver not installed, cable damaged
-1001	Scanner not found	
-1003	Checksum error in firmware file	firmware file is damaged
-1003	Checksum error in firmware file	
-1003	<b>ERR_CHECKSUM</b>	Checksum error in firmware file
-1004	<b>ERR_FPGA_BOOT</b>	Error while reading the firmware
-1004	Error while reading the firmware	firmware file is damaged
-1004	Error while reading the firmware	
-1005	<b>ERR_FPGA_LOCKED</b>	FPGA does not react to a reset
-1005	FPGA does not react to a reset	
-1006	EPP timeout error	check cable, check epp settings
-1006	EPP timeout error	
-1006	<b>ERR_IO_TIMEOUT</b>	EPP timeout error
-1007	<b>ERR_I2C</b>	I2C bus error
-1007	I2C bus error	
-1008	<b>ERR_COMMUNICATION</b>	generic communication error with hardware device
-1009	Light barrier not free during activation	check if there is dust in front of the light barrier
-1009	Light barrier not free during activation	
-1009	Slot_notfree	Light barrier not free during activation
-1010	<b>ERR_DEV_TEMP</b>	Temperature in the device is too high
-1010	Temperature in the device is too high	cool down the device
-1010	Temperature in the device is too high	

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-1011	ERR_TIMEOUT	Timeout during user interaction \todo germish
-1011	Timeout during user interaction \todo germish	
-1012	Images_found	There is still data in the FIFO during an open
-1012	There is still data in the FIFO during an open	
-1013	ERR_SERIAL_NO	Wrong serial number
-1013	Wrong serial number	
-1014	ERR_IMAGE_TOO_SMALL	Image file is too small
-1014	Image file is too small	
-1015	ERR_IMAGE_UNDER_EXPOSURE	Image exposure too low
-1015	Image exposure too low	
-1016	ERR_IMAGE_OVER_EXPOSURE	Image exposure too high
-1016	Image exposure too high	
-1017	ERR_INVALID_PIXELSIZE	Wrong pixel size
-1017	Wrong pixel size	
-1018	Image not sufficiently homogeneous	
-1018	Image_noisy	Image not sufficiently homogeneous
-1019	SettingChanged	settings have changed since last calibration
-1019	settings have changed since last calibration	
-1020	action not allowed in this context: reopen, restart, reconnect, unhandled error,...	try scanning and machine is in clean mode, press cleaning button again
-1020	action not allowed in this context: reopen, restart, reconnect, unhandled error,...	
-1020	ERR_INVALID_CONTEXT	action not allowed in this context: reopen, restart, reconnect, unhandled error,...
-1021	action not allowed in this context: reopen, restart, reconnect, unhandled error,...	try scanning and machine is in clean mode, press cleaning button again
-1021	action not allowed in this context: reopen, restart, reconnect, unhandled error,...	
-1021	ERR_NOT_CONNECTED	action not allowed in this context: reopen, restart, reconnect, unhandled error,...
-1022	ERR_NOT_INITIALIZED	scanner is still intializing
-1022	scanner is still intializing	wait, if it takes to long restart the machine, up to 2 min are ok
-1022	scanner is still intializing	
-1023	ERR_CAPTURE_DRIVER	os kernel driver for capture unit failed
-1023	os kernel driver for capture unit failed	
-1024	block buffer overflow (imagefind errors)	
-1024	ERR_IMAGE_DATA_PROCESSING	block buffer overflow (imagefind errors)
-1100	Cassette did not reach output sensor	check if a cassette is sticking inside the machine, remove the cassette
-1100	ERR_SCAN_TIMEOUT	Cassette did not reach output sensor

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-1100	ScanTimeoutSlot1	Cassette did not reach output sensor
-1100	ScanTimeoutSlot1	Cassette did not reach output sensor
-1101	Cassette did not reach output sensor	
-1101	ScanTimeoutSlot2	Cassette did not reach output sensor
-1102	Cassette did not reach output sensor	
-1102	ScanTimeoutSlot3	Cassette did not reach output sensor
-1103	Cassette did not reach output sensor	
-1103	ScanTimeoutSlot4	Cassette did not reach output sensor
-1104	Erase Lamp do not start	check if lamp is damaged, replace the lamp
-1104	Erase Lamp do not start \todo germish	
-1104	ERR_ERASOR	Erase lamp do not start
-1104	LampErrorSlot1	Erase Lamp do not start \todo germish
-1105	Erase Lamp do not start	check if lamp is damaged, replace the lamp
-1105	LampErrorSlot2	Erase Lamp do not start
-1106	Erase Lamp do not start	check if lamp is damaged, replace the lamp
-1106	Erase Lamp do not start	
-1106	LampErrorSlot3	Erase Lamp do not start
-1107	Erase Lamp do not start	check if lamp is damaged, replace the lamp
-1107	LampErrorSlot4	Erase Lamp do not start
-1108	ERR_EXIT_BLOCKED	Shuttlemotor blocked
-1108	ShuttleblockedSlot1	Shuttlemotor blocked
-1108	Shuttlemotor blocked	check if something is blocking the output mechanism from the VistaScan Perio
-1109	Shuttlemotor blocked	check if something is blocking the output mechanism from the VistaScan Perio
-1110	Shuttlemotor blocked	check if something is blocking the output mechanism from the VistaScan Perio
-1111	Shuttlemotor blocked	check if something is blocking the output mechanism from the VistaScan Perio
-1112	Cassette sensor is blocked	check if there is dust in front of the light barrier
-1112	ERR_EXIT_PEND	Exit cartridge sensor is blocked or two cartridges at exit
-1112	Exit cassette sensor is blocked or two cassettes at exit	remove the cassettes
-1112	SensorblockedSlot1	Cassette sensor is blocked
-1113	Cassette sensor is blocked	check if there is dust in front of the light barrier
-1113	SensorblockedSlot2	Cassette sensor is blocked
-1114	Cassette sensor is blocked	check if there is dust in front of the light barrier
-1114	SensorblockedSlot3	Cassette sensor is blocked
-1115	Cassette sensor is blocked	check if there is dust in front of the light barrier
-1115	SensorblockedSlot4	Cassette sensor is blocked
-1116	ERR_ENTRY_BLOCKED	Entrymotor blocked

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-1116	feed module is blocked or machine is in cleaning position	press cleaning button again, or exchange the feed module
-1117	<b>ERR_MALPOSITION</b>	Wrong position after action
-1117	Interface cover, not in the correct position	remove the interface cover and place it again, maybe there is somethin in front of the second light barrier
-1118	<b>ERR_OPENED</b>	Component still opened
-1118	Motor blocked (transport- or clampmotor)	check if something is blocking the motors or exchange the motors
-1119	<b>ERR_BLOCKED</b>	Component/Motor blocked
-1120	<b>ERR_TRANSPORT_BLOCKED</b>	Imageplate transport is blocked
-1120	Imageplate transport is blocked	check if something is blocking the transport or exchange the transport module, check the light barriere is dirty, maybe the light protection sleeve is in front of the 2nd light barrier,
-1121	<b>EER_PART_MISSING</b>	Any part of the hardware still missing
-1122	<b>ERR_REFPOS_NOTFOUND</b>	Reference position was not found
-1150	Erasing lamp overheated	cool down the machine
-1150	<b>ERR_LAMP_TEMP</b>	Erasing lamp overheated
-1151	24V Power for Bleaching failed	check if you are using the correct power supply, exchange the power supply
-1151	<b>ERR_LAMP_SUPPLY</b>	24V Power for Bleaching failed
-1152	<b>ERR_NET_CONFIG_IMPOSSIBLE</b>	No access to ethernet settings. No network connection or behind router
-1152	No access to ethernet settings. No network connection or behind router	
-1153	device component in error state because of error in subcomponent	restart the machine
-1153	<b>ERR_SUBSYSTEM</b>	device component in error state because of error in subcomponent
-1154	common hardware error in electronics	exchange the mainboard
-1154	<b>ERR_HARDWARE</b>	common hardware error in electronics
-1160	<b>ERR_PRISMASPEED_NOTREACHED</b>	pentaprisma did not settle at required speed
-1160	pentaprisma did not settle at required speed	exchange the beam deflector unit, also called pentaprism drive
-1170	<b>ERR_SOL_TIMEOUT</b>	Error at SOL signal / pentaprism Update firmware to 1752
-1170		exchange the SOL sensor
-1171		return to Dürr Dental
-1172		exchange of the beam deflector unit (can be done from a technician on-the-spot)
-2000	<b>ERR_RESTRICTED</b>	server process: insufficient permissions
-2000	server process: insufficient permissions	check network settings and permissions, switch of firewall to check if the firewall is blocking the communication
-2000	server process: insufficient permissions	VistaScan-Net
-2001	<b>ERR_NETWORK_CONNECT</b>	network connection could not be established
-2001	network connection could not be established	check network settings and permissions, switch of firewall to check if the firewall is blocking the communication
-2001	network connection could not be established	VistaScan-Net

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-2002	<b>ERR_RESTRICT_REENTRANT</b>	non reentrant function called twice
-2002	non reentrant function called twice	restart the machine
-2002	non reentrant function called twice	VistaScan-Net
-2003	INTERNAL Timed out waiting for operation to finish (mutex).	restart the machine
-2004	<b>ERR_NET_FORMAT</b>	RPC call or response is malformed
-2004	RPC call or response is malformed	check the network settings
-2004	RPC call or response is malformed	VistaScan-Net
-2005	call back function invalid	restart the machine
-2005	call back function invalid	VistaScan-Net
-2005	<b>ERR_INVALID_FUNC_PTR</b>	call back function invalid
-2006	<b>ERR_NO_SCANMODE_SELECTED</b>	No scanmode has been selected, but is required to continue
-2007	<b>ERR_DEVICEID_MISMATCH</b>	The device id in the imda.ini doesn't fit to the id of the connected device
-2300	Communication problems	check the network/USB settings, communication problems
-2346	<b>ERR_PROPERTY_TYPE_UNKNOWN</b>	trying to access unsupported property
-2346	trying to access unsupported property	check the network/USB settings, communication problems
-2346	trying to access unsupported property	netpp
-2347	<b>ERR_PROPERTY_SIZE_MATCH</b>	target memory insufficient for property request
-2347	target memory insufficient for property request	netpp
-2347	target memory insufficient for property request	check the network/USB settings, communication problems
-2490	<b>ERR_NETPP_CONNECTION_LOST1</b>	check the network/USB settings, communication problems
-2494	<b>ERR_NETPP_CONNECTION_LOST</b>	check the network/USB settings, communication problems
-2495	<b>ERR_NETPP_TIMEOUT</b>	check the network/USB settings, communication problems
-2496	Checksum failure	check the network/USB settings, communication problems
-2500	<b>ERR_NETPP_CONNECT</b>	check the network/USB settings, communication problems
-3000	<b>ERR_MECH_UNDEFINED</b>	The mechanic is in an undefined state and can not be recovered
-3000	The mechanic is in an undefined state and can not be recovered	MedScan
-3001	A motor does not behave as expected.	MedScan
-3002	<b>ERR_SENSOR</b>	A sensor did not trigger.
-3003	cartridge shutter is not fully open	MedScan
-3003	<b>ERR_SHUTTER</b>	cartridge shutter is not fully open
-3004	The cartridge tray does not work properly.	MedScan
-3005	<b>ERR_ARMCONTROL</b>	The arm controller does not work properly.
-3006	<b>ERR_ARMOPENPOS</b>	The entry-arm did not reach its 'open' position.
-3006	The entry-arm did not reach its 'open' position.	MedScan
-3007	<b>ERR_ARMTRAPPED</b>	The arm/the grabbers are trapped in the cartridge
-3007	The arm/the grabbers are trapped in the cartridge	MedScan

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-10001	Warning: There are still images in the image directory during an open	Warnungen
-10002	Filterscript insufficient permissions, Wrong magic number	Warnungen
-10002	<b>WARN_FILTER_RESTRICTED</b>	Filterscript insufficient permissions, Wrong magic number
-10003	End of erasor lamp life reached	Warnungen
-10003	<b>WARN_LIFETIME</b>	End of erasor lamp life reached
-10004	image has no valid meta information (resolution, ...)	Warnungen
-10004	<b>WARN_IMAGE_META_DATA_MISSING</b>	image has no valid meta information (resolution, ...)
-10005	GetPreviewLine() called without data available	Warnungen
-10005	<b>WARN_NO_PREVIEW_AVAIL</b>	GetPreviewLine() called without data available
-10006	image has invalid or corrupt meta information (resolution, ...)	Warnungen
-10006	<b>WARN_IMAGE_META_DATA_CORRUPT</b>	image has invalid or corrupt meta information (resolution, ...)
-10007	action makes no sense in this context - ignored: reclose, ,...	Warnungen
-10007	<b>WARN_INVALID_CONTEXT</b>	action makes no sense in this context - ignored: reclose, ,...
-10008	Device or laser temperature high	Warnungen
-10008	<b>WARN_DEV_TEMP</b>	Device or laser temperature high
-10009	One of the supply voltages is out of allowed range	Warnungen
-10009	<b>WARN_VOLTAGE</b>	One of the supply voltages is out of allowed range
-10010	scanner has invalid or corrupt data information (typically CRC error)	Warnungen
-10010	<b>WARN_DATA_CORRUPT</b>	scanner has invalid or corrupt data information (typically CRC error)
-10011	End of transport belt life reached	Warnungen
-10011	<b>WARN_MAINTENANCE_TRANSPORT</b>	End of transport belt life reached
-10012	error or warning in subcomponent, but device still working	Warnungen
-10012	<b>WARN_SUBSYSTEM</b>	error or warning in subcomponent, but device still working
-10013	<b>WARN_MAINTENANCE_CARTRIDGE</b>	End of cartridge life reached
-10100	Addressed device lacks some features	Warnungen
-10100	<b>WARN_MISSING_CAPABILITIES</b>	Addressed device lacks some features
-10200	directory name is valid but directory is not existing (yet)	Warnungen
-10200	<b>WARN_DIR_DOESNT_EXIST</b>	directory name is valid but directory is not existing (yet)
-12002	non reentrant function called twice	Warnungen
-12003	API mutex timeout - access to api failed	Warnungen
-12990	<b>WARN_POSSIBLE_UNPARSED_ENVIRONMENT_VARIABLE_FOUND</b>	
-90000	Power save mode has been activated	Warnungen